

Flaminal is tailor made for you.

User friendly wound treatment.

Exclusively available with EBOS Healthcare



Flaminal® is easy to apply for both the patient and health care professional making it ideal for self/shared care.



▶ STEP 1: CHOOSE

Clean the wound following local protocols and choose which Flaminal® to start with according to exudate levels.

Use **Flaminal® Hydro** on slightly to moderately exuding wounds

Use **Flaminal® Forte** on moderately to highly exuding wounds

▶ STEP 2: APPLY

Apply Flaminal® covering the entire wound bed with a sufficiently thick layer (4-5mm). Flaminal® can be applied in a number of ways:



Directly from the tube



Directly onto the dressing



With a syringe



With a nozzle




With a spatula

▶ STEP 3: COVER

Cover Flaminal® with the appropriate secondary dressing according to exudate levels.

Slight Exudate		Transparent film (polyurethane) or non-adherent dressing
Moderate Exudate		Transparent film (polyurethane) or non-adherent dressing or absorbent non-adherent dressing
High Exudate		Absorbent non-adherent dressing

FOR MORE INFORMATION CONTACT YOUR ACCOUNT MANAGER ON **FREEPHONE 1800 269 534** OR VISIT www.eboshealthcare.com.au

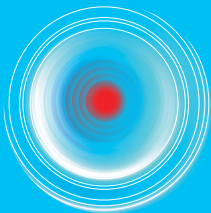

For a variety of wound types.

Due to it's unique composition

Flaminal® for acute and chronic wounds or wounds at risk of infection.

Exclusively available with EBOS Healthcare

INDICATIONS

- Leg ulcers
- Diabetic foot ulcers
- Superficial and deep partial thickness burns
- Post-surgical wounds
- Traumatic wounds
- Pressure ulcers
- Skin tears
- Wounds from radiotherapy
- Oncology wounds



Infected wounds



Granulating wounds



Sloughy wounds



Necrotic wounds

GOOD TO KNOW

If dry whitish alginate flakes appear in the wound:

- When using Flaminal® Forte, the wound is too dry. Step down to Flaminal® Hydro.
- When using Flaminal® Hydro, then ensure the wound is covered with a dressing that retains moisture. The flakes will disappear when the wound moisture balance is restored.

SHELF LIFE

Tubes: up to 12 months after first opening.

Tubs: up to 1 week after first opening.

ORDERING INFORMATION

EBOS Code	Description	UOM
AS14500	Flaminal Forte 50g	Each
AS14540	Flaminal Hydro 50g	Each
AS04515	Flaminal Hydro 15g	Each



Flaminal® Hydro



Flaminal® Forte

A different dimension of wound care.



FOR MORE INFORMATION CONTACT YOUR ACCOUNT MANAGER
ON FREEPHONE 1800 269 534 OR VISIT www.eboshealthcare.com.au

ESSENTIAL TO HEALTH