




### STEP 1

Please ensure you **wash your hands** thoroughly before and after application of Flaminal® Hydro / Forte.

**Cleanse your wound with clean water** or a specific wound cleansing product if advised by your nurse/doctor.

If you find you have **Flaminal® Hydro / Forte** from previous applications remaining on or in the wound that does not rinse off easily then this can be safely left in place.

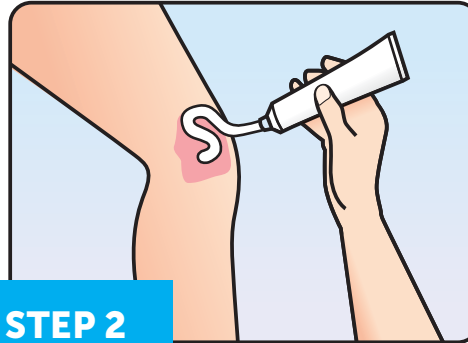


### STEP 2

After cleaning your wound, **apply Flaminal® Hydro / Forte with a thick layer of approx. 0.5cm.**

You can apply Flaminal® directly to your wound or use an applicator if you have been provided with one.

Try to avoid the opening of the tube or tube applicator coming in to direct contact with the wound.

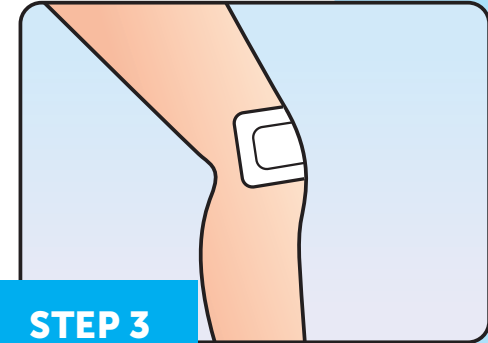

### STEP 3

**Cover your wound with an appropriate** dressing as advised by your nurse/doctor.

The dressing type you use will depend on the condition of your wound, is it wet or dry?

- For wet wounds use a non stick absorbent dressing.
- For drier wounds use a non stick dressing. If your wound starts to become wet or your dressing starts to leak after 24 hours use a non stick absorbent dressing.

**Change your dressing every 1-4 days** or when excessive leaking as advised by your nurse/doctor. See note.

### PLEASE NOTE:

In the first few days of treatment you may need to change your dressing more often every 1 – 2 days. As your wound improves you can reduce dressing changes to every 3 – 4 days as recommended by your nurse/doctor.

You have been prescribed Flaminal® Hydro and Forte by your Health Care Professional for use at home, please follow these guidelines for use.

**Please read the instructions for use carefully.**

## BENEFITS

- Helps to keep your wound clean
- Keeps your wound moist and in the right healing conditions
- Flaminal® Hydro / Forte is safe for the skin and wound tissue
- Protects the edges of your wound
- Offers protection against bacteria
- Reduces wound odour
- Helps wound healing

## CONTRAINDICATIONS

- Do not use if you have a known allergy to one of the components
- Do not apply to eyelids or in the eye. Should it come into contact with your eye, rinse the eye thoroughly

**Flaminal**



Your nurse/doctor will advise you to use Hydro or Forte depending on your wound type.

For dry to moderately wet wounds

Moderate to wet wounds

# Flaminal®

Contact EBOS Healthcare toll free number 1800 269 534.

Please find further information at [www.eboshealthcare.com.au](http://www.eboshealthcare.com.au)

If you have any questions relating to your care or wound then we advise that you contact your nurse or doctor.



| EBOS Code | Description        | UOM  |
|-----------|--------------------|------|
| AS14500   | Flaminal Forte 50g | Each |
| AS14540   | Flaminal Hydro 50g | Each |

**Flen Health**  
advanced skin healing

**EBOS**  
HEALTHCARE

ESSENTIAL TO HEALTH



**Flaminal**

## Self-care guide for patients

**EBOS**  
HEALTHCARE

ESSENTIAL TO HEALTH