

HydroClean® plus

Application guide



HydroClean® plus



HydroClean® plus is an All-In-One wound dressing that features an innovative rinsing and absorption mechanism for fast and effective wound healing.



Select the suitable size then open the dressing pack by peeling apart the top most tabs.



Position the unmarked side of the dressing directly onto the wound, the HydroClean logo should be visible after placement. The dressing should be slightly larger than the wound. A skin protectant may be used on the peri-wound area.




Use a suitable secondary dressing if necessary depending on exudate level e.g. Zetuvit Plus

HydroClean® plus cavity



HydroClean® plus cavity for deep wounds.

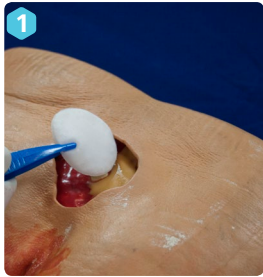

Select a suitably sized cavity dressing to pack the wound the dressing must be in direct contact with the wound bed.



Maintain optimal wound moisture by covering with a regular HydroClean® plus and tape in place.



Cover with a Zetuvit plus or similar dressing for heavily exuding wounds.



Where necessary secure dressing with tape. Where the tape crosses tight contours provide small cuts to accommodate for movement.

TIP
For low exudate levels substitute with a film dressing such as Hydrofilm, or Hydrofilm Plus.

Adhesive Fixation Options



HydroFilm® Transparent Film or Omnifix® Retention Tape

HydroFilm® is a self-adhesive transparent film dressing made of semi-permeable, waterproof polyurethane film. Omnifix® is self adhesive non-woven fixation fabric.



Peha-haft® Retention Bandage

Fixate with Peha-haft retention bandage. Peha-haft is a breathable fabric retention bandage which does not stick to skin.



Pütterbinde® Short-stretch Bandage

Compression therapy with Pütterbinde® short-stretch bandage. Pütterbinde is a short-stretch bandage for high compression.

HydroTac®

Application guide



HydroTac® Comfort



HydroTac® Comfort is a unique foam dressing with **AquaClear™ Technology**, combining absorption and moisture donation properties for healing efficiency with an adhesive border.



Select the suitable size then open the dressing pack by peeling apart the top tabs. The dressing is protected by four tabs labelled 1, 2, 3 & 4.



Peel away tabs marked 1 & 2 revealing the dressing's AquaClear™ layer. Place this exposed face to the wound, then carefully pull the tabs outwards to apply the dressing's adhesive border.



With tabs 1 & 2 fully removed, ensure that the adhesive layer is well adhered.



Peel away remaining tabs labelled 3 & 4, again ensuring that the adhesive border is well adhered.

HydroTac®



HydroTac® is a unique foam dressing with **AquaClear™ Technology**, combining absorption and moisture donation properties for healing efficiency.



Select the suitable size then open the dressing pack by peeling apart the top tabs. The dressing's AquaClear™ gel layer is protected by two tabs labelled 1 & 2.



Peel away tab number 1 revealing the AquaClear™ layer. Place the gel layer to the wound.



Peel away tab number 2 while applying the dressing to the wound surface.



Fixate the dressing with a breathable bandage such as Peha-haft®.

The versatility of HydroTac®



HydroTac® Comfort features an adhesive layer and is used for traumatic wounds, post-operative wounds, abrasion wounds, pressure injuries.



HydroTac® can be used for ulcers, pressure injuries, diabetic foot ulcers, traumatic wounds and frail, sensitive skin.

